

## INDICIN FOAM

### FOR LIGHT SOILING

Spray the object and let it work for 3-5 minutes



Medical instrument trolley

**INDICIN FOAM**  
CLEANING AND DISINFECTION



Indicin Foam

**DAILY USE FOR LIGHT SOILING**

Not for Kitchen use

There is no need to rinse



Use the sprayer



Let it work for 3-5 minutes



Don't rinse



Warning

## ALKLANET

### WITH NORMAL SOILING

Spray the product on a clean cloth




Medical instrument trolley

**ALKLANET**  
SPRAY CLEANER




Alklanet

**FOR NORMAL SOILING**

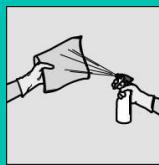

Spray on a clean cloth

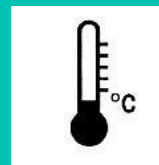

Cold

There is no need to rinse

## HELIOS BRILLANT

### WITH HEAVIER SOILING

Apply product to a soft sponge



Medical instrument trolley


**HELIOS BRILLANT**  
ABRASIVE




Helios Brilliant

**FOR HEAVIER SOILING**

There must be rinsed

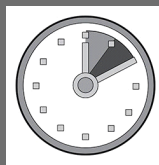


Let it work for 5-10 minutes

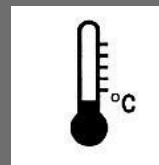

Cold

Do not use on plastic surfaces

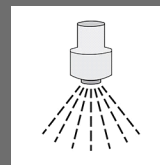

Rinse



Warning